



Detail 500ML Applicators

Water based fluorescent tail paint for detection of oestrus activity

- Available in 4 vivid fluorescent colours
- Quick and easy to apply for oestrus detection
- Use to follow the FIL Mating Management System
- Lasts up to 28 days subject to conditions and correct application

USEFUL INFORMATION

- Ensure animals are clean and dry before painting. Remove any loose hair.
- Apply a strip of paint 15cm long by 5cm wide forward along the spine from the tail head.
- Oestrus activity is indicated by the rubbing, or removal of the paint patch.
- Detail correctly applied will last up to 4 weeks on cows that have not cycled, depending on conditions. Cows which are penned or in close confinement may need to be touched up every 7-10 days.
- To use Detail to its full potential, follow the FIL Mating Management System:
- Paint all cows with RED paint three weeks after calving. If rubbed, this indicates when the cow has started to cycle again. If the cow has not come into heat after calving you can take remedial action.
- Paint all cows with GREEN paint immediately prior to commencing your AI programme. Rubbed paint identifies the cows coming into heat on a daily basis.
- Paint each cow with BLUE paint as soon as she has been AI'd. This will tell you if she has held to the service.
- Paint all cows with YELLOW when pregnancy is confirmed.
- Cows with RED paint have not cycled after calving
- Cows with GREEN paint are waiting to be AI'd
- Cows with BLUE paint are holding to AI service
- Cows with YELLOW paint are confirmed pregnant



Udder Mark 500ml

New 500ml Aerosol to indicate cows that require a milk with-holding period....

- New 500ml aerosol- RED (Stop) and Green (Go).
- The product is designed specifically for the skin of the udder.
- Will last on the udder for up to 14 days depending on weather conditions.
- The aerosol features an any-way actuator- meaning it can be applied from any angle
- The aerosol has a directional spray head which allows the farmer to clearly mark the treated quarter.

USEFUL INFORMATION

- Shake can well before use.
- Ensure udders are clean and dry before applying.
- Hold can approximately 15cms from the udder ensuring tail is clear of area to be sprayed.
- Apply UDDER MARK RED to cows treated with antibiotics.
- Overspray UDDER MARK GREEN when the cow has reached the recommended withholding period and milk is clear to return to the silo.

CONSUMABLES

CITO CUTTERS

- A push of a button cuts off the proper length of the sealed end of the semen straw.
- Easy to disassemble and washout without tools.

CITO THERMOMETERS

Essential for warm water method, measuring water temperature, until the desired temperature within safe zone is reached

CITO THAWING FLASKS

A thoroughly field-tested and proven thaw for properly thawing semen at a constant temperature controlled by accurate and sensitive electronic control. Made of tough non-corrosive plastic, the unit will take a lot of abuse, but at the same time protect all of its working elements. Indicator lights show when the unit is ready to use, and when it is working and maintaining water bath at proper temperature.



ESTROTECT™



EstroTECT™ is an economical, simple to use and highly effective heat detection aid. It features simple “rub-off” technology for superior results in your visual heat detection program. Available in five fluorescent colours – red/orange, green, yellow, fuchsia and blue – EstroTECT™ is a management tool for artificial insemination programs or monitor recipients for embryo transfer. Its simple design means there is no messy glue. Just peel off the backing

paper and stick it on the cow. When cows are mounted during standing heat, the scratch off surface of the EstroTECT™ patch is rubbed off to expose the brilliant coloured layer underneath. The brilliant colour of the activated EstroTECT™ is highly visible in the paddock and in poor light conditions.

ESTROTECT™ LEG MARKERS

Whether for dry cows, medical treatment, sorting or any other reason, ID by EstroTECT™ is a simple and effective method to identify your animals. ID by EstroTECT™ self-adhesive bands are highly visible and easy to apply. Available in five fluorescent colours, the applications for ID by EstroTECT™ are endless.

FIL TELL TAIL: OIL BASED FLUORESCENT HEAT DETECTION TAIL PAINT



- Available in 6 vivid colours
- Quick and easy to apply for oestrus detection
- Lasts up to 28 days subject to conditions and to correct application

FIL UDDERMARK: FLUORESCENT AEROSOL ANIMAL MARKER TO INDICATE COWS THAT REQUIRE A MILK WITH-HOLDING PERIOD

- Specifically formulated to last for up to 14 days on the udder
- 360 degree anyway valve for ease of application
- Available in red (stop) and green (go)
- Fluorescent colours easy to see in poor light

KAMAR

- Helps you catch night heats and short HEATS that occur when you're not watching.
- Helps you DISTINGUISH between standing heat and FALSE mounting behaviour.
- Available in 25's and 100's



NITROGEN FLASKS

CRV Xseed can supply you with a wide range of new and used nitrogen flasks. Contact your local representative or our office for more information and quotes.

NJP LINIMENT

- The special 12 hour active formula is an effective udder care with Japanese peppermint oil.
- It supports and protects the natural resistance of stressed udder tissue for long periods.
- The mild formula with udder skin care also makes the udder skin supple.
- There is no waiting time for original NJP Liniment and it has the DLG quality mark.



DAIRYCIDE

Dairycide ADE 5l & 20l
 Dairycide Plus 5l & 20l
 Dairycide Teat wash 5l & 20l
 Dairycide Dairy dine 20l
 Dairycide X 5l & 20l
 Dairycide 10 5l & 20l
 Dairycide Barrier cream
 Dairycide Handsoap
 Dairycide Teat seal
 Dairycide Wipes

OTHER PRODUCTS

Gloves
 Milking gloves latex
 Lube
 Mop caps
 Pistolettes (mini/medium/combo)
 Propylene Glycol 25l
 Sanitary sheaths
 Sheaths
 Thermometer

CRV XSEED SUPPLIES A WIDE RANGE OF CONSUMABLES. A COMPLETE LIST OF CONSUMABLES CAN BE FOUND AT OUR WEBSITE WWW.CRV4ALL.CO.ZA